



Ready to use impregnated wipes for disinfecting surfaces.

perform® classic wipes EP

Our Plus

- fast, effective and residue free
- economical with 200 jumbo wipes (approx. DIN A4 size) per box
- colour free and perfume free
- efficacy tested according to European norms (EN)

Application areas

For disinfection of surfaces in areas where fast microbiological effectiveness is required.

Instructions for use

Tear off wipes downward at an angle. Thoroughly wipe surfaces with perform® classic wipes EP, ensure complete wetting and allow to dry. The product dries off quickly. Close the lid tightly after use. Do not use on surfaces that are particularly sensitive to alcohol, such as acrylic glass.

Microbiological efficacy

| Efficacy | Concentration | Contact time |
|--|---------------|--------------|
| bactericidal <i>EN1276, EN13697</i> | ready-to-use | 1 min. |
| fungicidal <i>EN1650, EN13697</i> | ready-to-use | 1 min. |
| enveloped viruses (incl. HIV, HBV, HCV) <i>in accordance with DVV (German Registered Association for Combating Viral Diseases)//RKI Guideline</i> | ready-to-use | 1 min. |



perform[®] classic wipes EP

Product data

One box contains: 200 ready to use impregnated wipes.
100 g active agent solution contains:
25 g Ethanol (94 %), 35 g Propan-1-ol.

Chemical-physical data

| | |
|--------------------|------------------------------------|
| Color | colourless |
| Density | ca. 0,89 g/cm ³ / 20 °C |
| Flash point | 27 °C / Method : DIN 51755 Part 1 |
| Form | liquid |
| pH | Not applicable |
| Viscosity, dynamic | not determined |

Special advice

Use disinfectants safely. Always read the label and product information before use.

Information for order

| Item | Delivery form | Item no. |
|------------------------------------|---------------|----------|
| perform classic wipes EP 200 ST DS | 10/Carton | 70000186 |

These products are not available in every country. For more information please contact our local subsidiary or distributor.

Environmental information

schülke manufactures products economically and with advanced, safe and environmentally friendly production processes while at the same time maintaining out high quality standards.

Expert opinion and information

Please visit our website for an overview of all available literature/reports on the product: www.schuelke.com.

For individual questions:

Customer Sales Service

Phone: +49 40 52100-666

E-Mail: info@schuelke.com



CEPRUFTES
UMWELTMANAGEMENT
REG. NR. D-150-00003



Schülke & Mayr GmbH holds a Manufacturer's Authorisation according to sect 13 para 1 German Drug Law and Certificates of GMP Compliance for medicinal products.



A company of the Air Liquide-Group

schülke Headquarters
Schülke & Mayr GmbH
Robert-Koch-Str. 2
22851 Norderstedt, Germany
Phone +49 (0) 40 - 52100 - 0
Fax +49 (0) 40 - 52100 - 318
www.schuelke.com
mail@schuelke.com